

Product name: **Novel Coronavirus (COVID-19) Antigen Test Kit (Colloidal Gold) nasal**
 Catalogue No. 303036
 Referred to as: Lyher COVID-19 antigen test kit (nasal)

Manufacturer: Hangzhou Laihe Biotech Co., Ltd., Hangzhou, Zhejiang, China

General description of the test and how it works


The Lyher COVID-19 Test Kit is a single-use test kit intended to detect the novel coronavirus SARS-CoV-2 that causes COVID-19. This test is authorized for prescription home use with self-collected nasal swab samples in individuals aged 18 and older and in individuals aged 18 and under when the specimen is collected by his/her guardian.

This test utilizes an immune colloidal gold technique for the detection of SARS-CoV-2 antigens (N-protein). Positive results are indicative of the presence of SARS-CoV-2.

Intended use of the test, including type of sample/sampling method and description of the testing target group.

This Lyher COVID-19 antigen test will be used as self-test for at home rapid testing of SARS-CoV-2 infection. This product's intended use is for qualitatively testing novel coronavirus antigens in human nasal specimens. For this a nasal swab is taken by inserting a swab into one nostril. The swab tip should be inserted up to 2-4 cm until it touches the inside wall of the nostril. The swab needs to be rolled 7 times inside the nostril after which the swab is withdrawn. The testing target group is the general public.

Clear (digital) illustrations or photos of the various components of the test and photos of the packaging from all sides and of the labels.

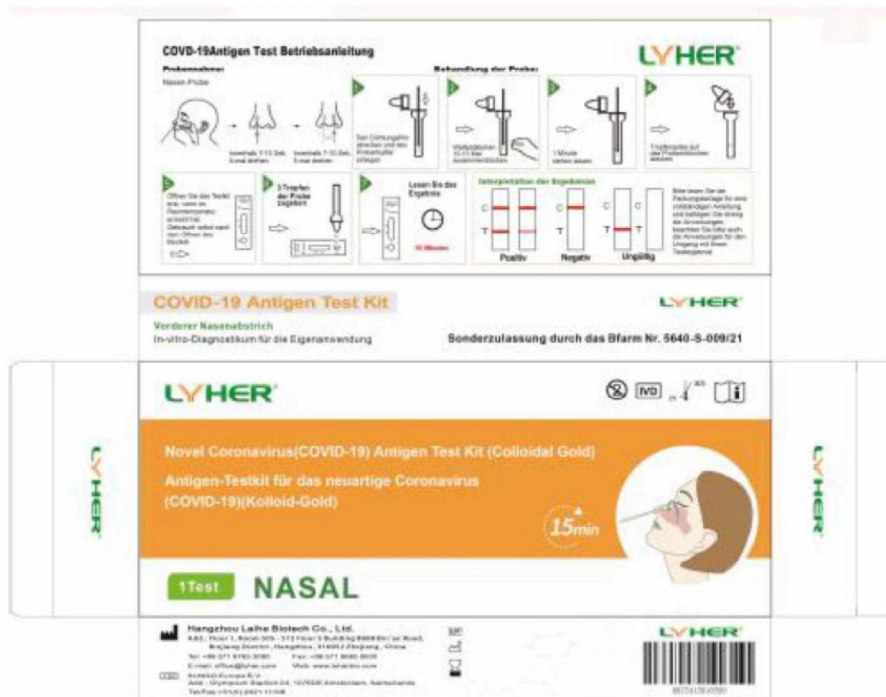

Packaging box from all sides



package test device



test device



paper



Nasal swab device



prepacked extraction buffer