

80mm



190mm

100mm



50 PCS

SURGICAL
FACE MASK


EN 14683 Type IIR



SURGICAL
FACE MASK

50 PCS

EN 14683 Type IIR

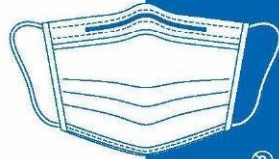

SURGICAL FACE MASK
EN 14683 Type IIR

TopCare & Omunda Medical Device Co., Ltd.
Fourth floor, 88th Building No.1, Dazhong Road 5, Donghua Industrial
Zone, Minsheng Town, Baoyan District, Changshu City, Jiangsu
215163, China

Omunda Medical Technology Service GmbH
Von Oppen Weg 15, 14476 Potsdam, Germany
Tel: +49 30 9239 9627 Email: omunda@omunda.com

Distribution:
18500100001 | Phone: +86 21 6057 4677
18500100002 | E-Mail: omunda@omunda.com
18500100003 | E-Mail: omunda@omunda.com | Fax: +86 21 6057 4678

2020-05

2022-04


注册证号: 国械注进202040955
生产许可证: 苏械注准2020020218



SURGICAL
FACE MASK

50 PCS

EN 14683 Type IIR

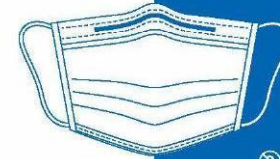

Warnings:
1. Please store this product in a clean, sanitary, ventilated and dry place, and do not store with toxic or
corrosive goods. Do not put sharp objects together to avoid scratches.
2. Check whether the package is damaged before use. If there is any damage, please do not use
the product if possible. Please do not reuse it.
3. Do not exchange with others.
4. Please keep it properly. Do not allow infants to play with it. Children should use it under the
guidance of adults.
5. This product cannot be washed. If the mask is damaged or considered unhygienic, discard and
replace it.
6. In order to ensure good filtration efficiency, it is recommended to replace the mask every 3-5 hours.

Instructions:
1. Take out the mask and put it on, pull the flap out until it completely covers the bridge of the nose
and chin.
2. Press down the two sides of the malleable nose piece to match the bridge of the nose and adjust
the mask to fit to the whole face shape.
3. It is recommended to replace the mask every 3-5 hours.
4. When removed, do not touch the front or the back of the mask (contaminated surface).